



# Fuzion™ Cleaner Disinfectant



EPA-registered to kill *C. diff* spores and 35 other pathogens in 2 minutes or less

## Bleach efficacy that's easy on surfaces

Clorox Healthcare® Fuzion™ Cleaner Disinfectant combines the proven power of bleach with excellent aesthetics for broad use throughout the facility.

- ✓ **Kills *C. diff* in 2 Minutes:** Hypochlorous acid solution is a strong oxidizer and highly effective surface disinfectant.
- ✓ **Broad Surface Compatibility:** Safe for use on stainless steel, clear plastics, finished wood and many other common surfaces.
- ✓ **Low Residue:** Leaves surfaces looking clean, not cloudy.
- ✓ **Low Odor:** Mild odor dissipates in minutes.
- ✓ **Easy to Use:** No PPE required.

Fuzion™ at a Glance	
<i>C. difficile</i> contact time	2 min
TB contact time	1 min
Bacteria, virus and fungi contact time	1 min
Kills Norovirus?	Yes
Signal word	Caution
HMIS rating	0-0-0
PPE required	None

# Clorox Healthcare® Fuzion™ Cleaner Disinfectant Spray

	Effective Against:	Kill Time	
Bacteria	<i>Acinetobacter baumannii</i>	1 min	
	<i>Burkholderia cepacia</i>	1 min	
	<i>Campylobacter jejuni</i>	1 min	
	Carbapenem-resistant <i>Klebsiella pneumoniae</i>	1 min	
	<b>Clostridium difficile spores</b>	<b>2 min</b>	
	<i>Enterobacter aerogenes</i>	1 min	
	<i>Enterobacter cloacae</i> New Delhi Metallo-Beta Lactamase-1	1 min	
	<i>Enterococcus faecalis</i>	1 min	
	<i>Escherichia coli</i> O157:H7 ( <i>E. coli</i> )	1 min	
	<i>Klebsiella pneumoniae</i> New Delhi Metallo-Beta Lactamase-1	1 min	
	Methicillin-resistant <i>Staphylococcus aureus</i> (MRSA)	1 min	
	Multidrug-resistant <i>Staphylococcus aureus</i>	1 min	
	Multidrug-resistant <i>Streptococcus pneumoniae</i>	1 min	
	Multidrug-resistant <i>Enterococcus faecium</i>	1 min	
	<i>Proteus mirabilis</i>	1 min	
	<i>Pseudomonas aeruginosa</i>	1 min	
	<i>Salmonella enterica</i>	1 min	
	<i>Serratia marcescens</i>	1 min	
	<i>Staphylococcus aureus</i>	1 min	
	<i>Staphylococcus epidermidis</i> (CoNS)	1 min	
<i>Strenotrophomonas maltophilia</i>	1 min		
Vancomycin intermediate-resistant <i>Staphylococcus aureus</i> (VISA)	1 min		
Vancomycin-resistant <i>Enterococcus faecalis</i> (VRE)	1 min		
Viruses	Avian Influenza A (H5N1) Virus	1 min	
	Hepatitis B Virus (HBV)	1 min	
	Herpes Simplex Virus Type I (HSV-1)	1 min	
	HIV Type 1	1 min	
	Human Coronavirus (RSV)	1 min	
	Influenza A Virus (H1N1)	1 min	
	Norovirus	1 min	
	Respiratory Syncytial Virus (RSV)	1 min	
	Rhinovirus	1 min	
	Rotavirus	1 min	
	Fungi	<i>Candida albicans</i>	1 min
		<i>Trichophyton mentagrophytes</i> (Athlete's Foot Fungus)	1 min
	TB	<i>Mycobacterium bovis</i> (TB)	1 min

**Total: 36 Claims**


**Use as directed on hard, nonporous surfaces.**

## Excellent surface compatibility

Using Fuzion™ eliminates the need to make tradeoffs between efficacy and surface compatibility. Fuzion™ is a pH-neutral formula that won't cause damage to common surfaces. It can be safely used on stainless steel, chrome, countertops, sinks, mattress covers, glazed tiles—even clear plastics and glass.

In outside laboratory testing, Fuzion™ left significantly less residue on a clear acrylic surface compared to a quat/alcohol and a peracetic acid/hydrogen peroxide disinfectant:

**Residue Profile — Clear Acrylic**

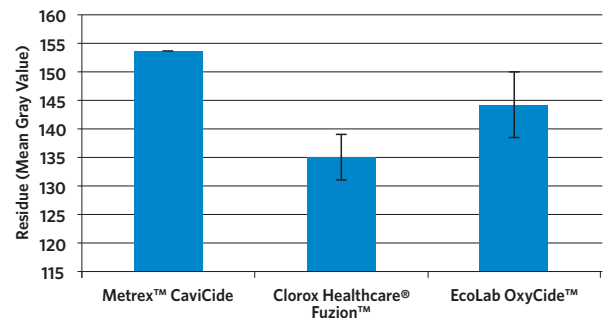

Fuzion™ enables facilities to use bleach in more places, providing strong protection against the spread of pathogens on surfaces.

### Product Information:

**Clorox Healthcare® Fuzion™  
Cleaner Disinfectant**  
EPA Reg. No. 67619-30

**Spray Bottle, 32 oz., 9/case**  
**Item No. 31478**

**Case UPC No. 10044600314782**



**HEALTHCARE®**

For more information, contact your Clorox sales representative or call **800-234-7700**.

email: [healthcare@clorox.com](mailto:healthcare@clorox.com)

visit us: [www.cloroxhealthcare.com](http://www.cloroxhealthcare.com)

© 2016 Clorox Professional Products Company,  
1221 Broadway, Oakland, CA 94612.